## Aeris® Balloon Inflation Device

## PREPARATION INSTRUCTIONS





- Open sterile package and remove the Aeris® Balloon Dilation Catheter
- Lift off the green protective sheath, revealing the balloon. Place protective sheath off to the side.
- If physician's preference is not to use stylet, remove and place off to the side



- · Remove the Aeris® Inflation Device from it's packaging
- · Position the lever (on the inflation device) to the left
- Pull back the plunger to fill the barrel with saline
- Invert and purge air, leaving fluid level in barrel:

10cc: 5mm, 6mm, 7mm, 8mm, and 9mm Balloons20cc: 10mm, 12mm, 14mm, 16mm and 18mm Balloons



## WHEN AERIS® BALLOON DILATION CATHETER IS IN POSITION TO INFLATE

- Remove stylet (if not previously removed)
- Attach balloon dilation catheter luer lock to the connecting tube of inflation device





 While maintaining pressure on the plunger, move lever to the right midline position

 Turn the plunger clockwise to the desired atm setting, matching hub color to the teardrop on inflation gauge.

• Inflate the balloon with saline to appropriate atm of pressure:

LUER LOCK COLOR ATM BALLOON SIZES

White Luer Lock 17 atm 5mm, 6mm, 7mm, 8mm, 9mm & 10mm
Yellow Luer Lock 10 atm 12mm, 14mm, 16mm & 18mm







YELLOW 10 atm

AFTER PROCEDURE IS COMPLETE

Unlock lever to the left, releasing pressure in the balloon. Pull back on plunger all the way and lock lever to the right midline position.